

Instructions for Use

Moisturising and Comfort Eye Drops (Vit A&E)

The Moisturising and comfort eye drops containing Vitamin A & E are intended for consumers who require an over-the-counter eye preparation to moisturise the eye and provide relief to uncomfortable eyes and is intended to provide supplemental moisture to the eye to provide comfort to dry, tired, irritated and/or strained eyes.

The moisturising and comfort eye drops are indicated for irritated, dry, tired and sensitive eyes that are uncomfortable due to occupational and environmental factors or irritations associated with allergies, dust, pollution and smoke.

- Uncomfortable eye symptoms can also be described by consumers as itchy, gritty, dry, burning, tearing, 'feeling like something is in your eye', needing to blink more frequently and / or sensitivity to light due to eye strain.
- Examples of occupational and environmental factors are computer use, driving, dry air / air conditioning, dusty environments and chlorinated water following swimming.

The product can be used by Adults (Children younger than 16 may use the product under supervision)

DESCRIPTION

The product is a clear solution, supplied in 10 ml /15 ml HDPE Container with nozzle and cap

MODEL DETAILS

Formulation code: Formulation F

Formulation F (Moisturising and Comfort drops with Vitamin A&E)

INGREDIENTS

0.1% Carmellose, 0.05% sodium hyaluronate, 1% polysorbate 80, 0.015% retinyl palmitate (Vitamin A) and 0.015% tocopherol acetate (Vitamin E)

DIRECTIONS FOR USE

- Ensure your hands are clean and dry before use
- Shake before use
- Check the bottle seal is not broken or missing before use. To open the bottle: Twist the cap anti-clockwise to break the seal.
- Tilt your head backwards and gently squeeze 1 to 2 drops into the eye
- Blink several times to cleanse the eye
- Repeat for the other eye as necessary.
- Replace cap after use.
- May be used prior to and following contact lens wear.
- Use as often as required / safe for everyday use
- Compatible with contact lenses

WARNINGS:

The product is sterile until first opened

- Do not use if tamper-evident seal is missing or broken.
- Do not swallow the solution
- If eye irritation occurs, discontinue use and consult an eye care practitioner. If condition worsens, consult a specialist.

- If you have been diagnosed with a medical eye condition or are using other eye medication, consult a healthcare professional before use.
- Do not use if allergic to any of the ingredients.
- May cause temporary blurring of vision, if affected do not drive or perform skilled tasks until vision is clear.
- For use in the eyes only
- This product should not be used to soak or disinfect contact lenses
- To avoid contamination, do not touch the dropper tip.
- Use within 90 days of first opening / Once opened, discard after 90 days
- Do not use after the expiry date

In the event of a serious incident occurring during the use of this medical device or as a result of its use, please report the incident to the manufacturer and the Competent Authority of the Country in which you are located

KEEP OUT OF SIGHT AND REACH OF CHILDREN.

STORAGE

Store between 5 °C and 30 °C

Avoid direct sunlight.

Do not use if package is open or damaged



East Midlands Pharma Ltd
Unit 1 Rothley Lodge, Loughborough Road
Rothley, Leicestershire, LE7 7NL, UK



Advena Ltd, Tower Business Centre,
2nd Flr, Tower Street, Swatar, BKR 4013
Malta



2797

For any additional questions on the use or safety of this product, please contact us on technical@empharma.co.uk

Revision History

Rv	Date	Description
01	18 Aug 2025	New electronic Instructions for Use