

Instructions for Use

Irritated Eye Drops (IRRIT)

The Cleaning drops containing Herbal extracts (Hamamelis & Calendula) can be used to cleanse and/ or rinse debris, irritating particulates and chemicals from the eye to cool, soothe and provide long lasting and instant relief from minor irritation.

Cleansing eye drops are pH balanced to the eye's tear fluid which means it's gentle on the eyes and is compatible with contact lenses. The drops help eyes feel more comfortable.

The intended purpose of the product is to provide additional volume of isotonic solution to cleanse, moisturize and provide relief to uncomfortable eyes. It can be used for irritated, dry, tired and sensitive eyes that are uncomfortable due to occupational and environmental factors or irritations associated with allergies, dust, pollution and smoke.

DESCRIPTION

The product is a clear solution, supplied in 10 ml /15 ml HDPE containers with a nozzle and cap .

MODEL DETAILS

Formulation code: IRRIT

Formulation B (Cleansing Drops)

INGREDIENTS

Sodium chloride 0.75%, calendula extract and hamamelis extract, disodium EDTA and polyhexanide 0.0001% in a borate buffer 0.125%.

DIRECTIONS FOR USE

- Ensure your hands are clean and dry before use
- Shake before use
- Check the bottle seal is not broken or missing before use. To open the bottle: Twist the cap anti-clockwise to break the seal.
- Tilt your head backwards and gently squeeze 1 to 2 drops into the eye
- Blink several times to cleanse the eye
- Repeat for the other eye as necessary.
- Replace cap after use.
- May be used prior to and following contact lens wear. May be used prior to and following contact lens wear.
- Use as often as required / safe for everyday use
- Compatible with contact lenses

WARNINGS:

- The product is sterile until first opened
- Do not use if tamper-evident seal is missing or broken.
- Do not swallow the solution
- If eye irritation occurs, discontinue use and consult an eye care practitioner. If condition worsens, consult a specialist.
- If you have been diagnosed with a medical eye condition or are using other eye medication, consult a healthcare professional before use.
- Do not use if allergic to any of the ingredients.

- May cause temporary blurring of vision, if affected do not drive or perform skilled tasks until vision is clear.
- For use in the eyes only
- This product should not be used to soak or disinfect contact lenses
- To avoid contamination, do not touch the dropper tip.
- Use within 90 days of first opening / Once opened, discard after 90 days
- Do not use after the expiry date

This product should not be used on children younger than 2 years without medical advice as it contains boron and may impair fertility in the future.

In the event of a serious incident occurring during the use of this medical device or as a result of its use, please report the incident to the manufacturer and the Competent Authority of the Country in which you are located

KEEP OUT OF SIGHT AND REACH OF CHILDREN.

STORAGE

Store between 5 °C and 30 °C

Avoid direct sunlight.


Do not use if package is open or damaged



East Midlands Pharma Ltd
Unit 1 Rothley Lodge, Loughborough Road
Rothley, Leicestershire, LE7 7NL, UK



Advena Ltd, Tower Business Centre,
2nd Flr, Tower Street, Swatar, BKR 4013
Malta

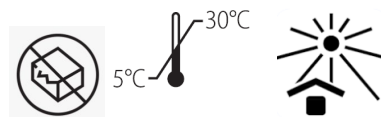

Revision History

Rv	Date	Description
01	18 Aug 2025	New electronic Instructions for Use